

Instructions For Pinworm Test

Please read instructions completely before you begin.

Broadlands Farm
101 Durkar Low Lane
Durkar
Wakefield
WF4 3BJ
01924 242851

This test should be completed between the hours of 3am and 5am. The specimen should be obtained before rising or a bowel movement.

1. Holding the cap, remove paddle from the container. Do not touch the paddle.
2. Separate the buttocks and press the sticky surface of the paddle against the anal area.
3. Return the paddle to the container and secure cap tightly.
4. Wash hands thoroughly with soap after you have finished.
5. Print patient information on the label.
6. Store specimen in a cool place until mailing back to PCI Europe in the container it arrived in for analysis.
7. Results will be made available within 7-10 working days.

Please send test kit back in original packaging

If you have any questions please contact us on enquiries@parasitetesting.co.uk.